Instructions for Use English from page 03

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Page 9 to 26 | Operation

How to use Lady-Comp.

Page 27 to 34 | Technical notes and warranty

What else you need to know.

About Lady-Comp

Lady-Comp calculates a woman's daily menstrual cycle status by interpreting her basal body temperature. Lady-Comp displays your menstrual cycle status using colored display screens with symbols. Lady-Comp is to be used to facilitate conception and track your cycle. Lady-Comp is not a replacement for contraception, nor is it a medical thermometer.

Included in delivery:

- Lady-Comp
- Sensor cable (170 mm)
- 2 AAA batteries (included, already in place)
- Micro USB cable (250 mm)
- Instructions for Use
- Quick Guide

Precautions

- Before using your Lady-Comp for the first time, please read these instructions carefully and keep them in a safe place for later reference.
- Before you start using your Lady-Comp, you need to entirely discontinue hormonal methods of birth control.
- Lady-Comp is not suitable if you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- Important: The first bleeding after stopping the pill or other hormonal contraceptive methods is a hormonal withdrawal bleeding and not a menstruation. DO NOT enter this bleeding as a menstruation.
- Lady-Comp does not protect you from sexually transmitted diseases.
- Never chew on the sensor.
- Take care that the appliance or parts thereof are not swallowed.
- All serious incidents associated with this product should be reported to the manufacturer and the relevant government authority responsible in the EU member state.
- **WARNING:** The device must not be modified.

- **WARNING:** Avoid using Lady-Comp at or less than 30 cm away from other electrical equipment, as this could result in improper operation.
- **WARNING:** Use of accessories, cables or equipment other than those specified in this document may cause electrical interference or lead to failure of Lady-Comp.
- Do not expose your Lady-Comp to high temperatures (e.g. boiling water).
- Do not expose your Lady-Comp to direct sunlight for long periods of time.
- Do not expose your Lady-Comp to liquids such as running water.
- Only take measurements in your mouth.
- Do not insert the measuring sensor too far into your mouth.
- Do not take measurements if the housing, the cable or the measurement sensor is broken.
- Do not take measurements if you have an injury in your mouth.
- Do not expose your Lady-Comp to heavy mechanical loads.
- Only take measurements when the USB cable is NOT connected.

- Do not stick sharp objects into your Lady-Comp.
- If your Lady-Comp causes irritation to your skin, we recommend you stop using it immediately and contact our Customer Support as soon as possible.
- Lady-Comp is a personal device. It must not be shared with another person. If you want to share Lady-Comp with a friend, you have to delete your data first.
- Only pass on Lady-Comp to third parties complete with the instructions for use. You will also find the instructions and further useful information available for download at *lady-comp.com/start*
- As the user, you have access to every feature of your Lady-Comp. You may also clean the sensor yourself, when your Lady-Comp is switched off.
- Your Lady-Comp is not a contraceptive.
- Keep Lady-Comp away from children and pets.

USB: To read out the data, you should only use devices that are capable to read a USB mass storage device. The USB port may only be used for connection to such devices. Lady-Comp cannot be used when it is connected via USB.

All the features at a glance

Sensor cable

Sensor holder


Display

Plus button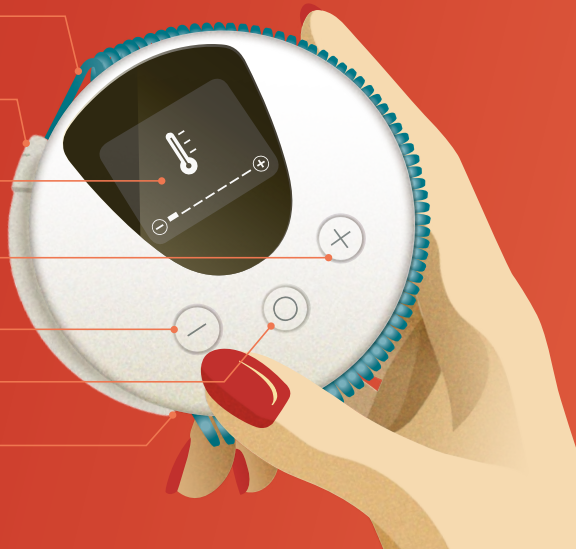

Minus button

Circle button


Measuring sensor

The button functions

-  Switch on / select menu / confirm
-  Navigate to the right / make a selection / increase value
-  Navigate to the left / cancel selection / reduce value
-  +  Press simultaneously:
Return to the higher menu level
-  3 Sec In the fertility menu:
opens input field for entering menstruation /
sexual intercourse

The menu levels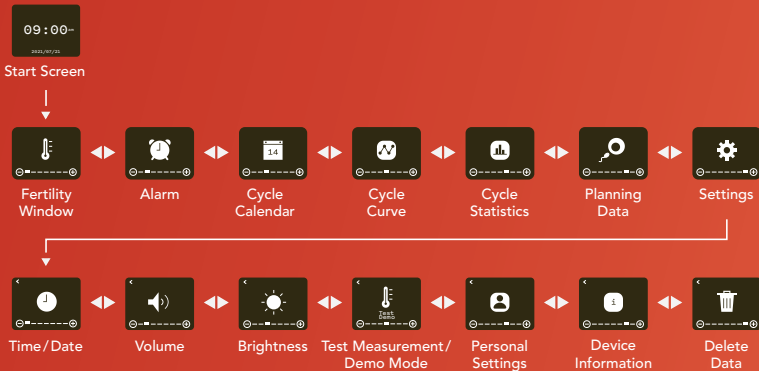

 Demo mode can only be used before the first measurement

Step 1:

Activate your Lady-Comp

1. Remove the strap  from the device.
2. Set the alarm for the next morning to activate your Lady-Comp. To do so, please press one of the 3 buttons and press the circle button to open the menu. Then, press the plus button to get to the alarm clock icon and press the circle button to select. Then, set the alarm time with the help of the plus and minus buttons. Confirm the entry for each field by pressing the circle button. The alarm clock is activated when you enter a checkmark in the last field by pressing the plus button and confirmed with the circle button. An alarm clock symbol will appear in the upper right corner of the initial time / date screen.
Your first measurement must be taken +/- 3 hours from your set alarm time.
3. You can now use Lady-Comp – but a measurement is only possible the next day. After the first successful measurement, you can also measure without the alarm.

i *If you have accurate records of your menstrual data, you can add them the day before the first measurement. After the first measurement, this is no longer possible. You can switch from degrees Celsius to degrees Fahrenheit in the personal settings. This will also change the date display (month/day).*

Adding historical menstruation data (optional)

1. Navigate to the fertility window and use the minus button to select the day to edit menstruation. Start by entering menstruation data that is furthest back in time.
2. Press and hold the circle button for 3 seconds.
3. Enter the menstruation in the input field with the plus button. With the minus button you can cancel the entry.
4. Press the circle button once to confirm the entry. The menstruation will automatically be entered for the next 2 days.


 *Incorrectly entered data that is more than 3 days in the past cannot be corrected.*

Step 2: Measure your temperature

1. To take your first measurement, or if you also use the alarm function in the future, simply deactivate the alarm tone by pressing one of the 3 buttons and then press one of the 3 buttons again. If the alarm is not used after the first measurement, press one of the 3 buttons to turn on the Lady-Comp and then press one of the 3 buttons again to start the measurement.
2. Place the measuring sensor as far back as comfortable to the left or right under your tongue and close your mouth. The measuring sensor should not be removed during the measurement. The device measures your temperature as soon as a flashing thermometer and a temperature indicator (initially without a value) appear on the display.
3. At the end of the successful measurement, you will hear a beep and the measured temperature and fertility status, which is valid until midnight, will be displayed. You can see your fertility status later in the day by the color of the start screen. Details of the last measurement can be found in the fertility window. On the next day Lady-Comp switches to the fertility forecast before you measure again.

i *If no measurement value can be determined after 3 minutes, the measurement is cancelled. You can cancel the measurement manually by pressing the plus and minus button simultaneously.*

The fertility window

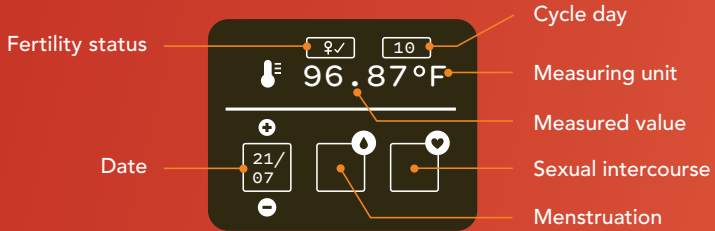What the screen colors mean:

not fertile

fertile /
possibly fertile

possibly fertile
(learning phase or cycle fluctuation)

Step 3: Enter your menstruation

If you see a question mark in the input field for menstruation after measuring, it means that Lady-Comp is asking you for your menstruation input.

- **If you are menstruating:** Press and hold the circle button for 3 seconds to open the input field and enter your menstruation by pressing the plus button. Confirm the entry by pressing the circle button. Enter your menstruation for each day of active bleeding and flow for a minimum of 3 consecutive days.
- **If you are not menstruating:** You do not have to do anything, just measure your temperature.
- **If you are menstruating, but you do not see a question mark:** Press and hold the circle button for 3 seconds. Enter your menstruation with the plus button and confirm the entry by pressing the circle button twice.

Overview: Displays of the fertility window

Cycle day flashes:
Ovulation (prediction)

Fertility status:
Not fertile

Fertility status:
Fertile/possibly fertile

Fertility status:
Highly fertile

Fertility status: Unknown,
but possibly fertile

Menstruation query

Menstruation confirmed/
not confirmed

Sexual intercourse confirmed/
not confirmed


Display lights up bright blue:
Possibly pregnant

Display lights up blue:
Probably pregnant


Temperature and Fertility: The course of your cycle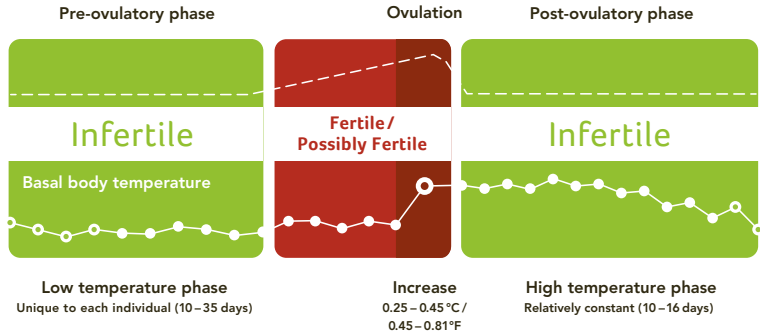

How to use your Lady-Comp correctly



- Make sure to take your temperature every day immediately after waking up, before you get up and become active. For optimal use of Lady-Comp, we recommend that you have at least 1 hour of continuous sleep before the measurement. If you have already gotten up, skip the measurement.
- If you are menstruating, enter this input accordingly. It is important to start with the first day of full blood flow.
- The more regularly you take your temperature, the faster Lady-Comp will get to know your cycle.
- When you start using your Lady-Comp, you will initially see a lot of yellow and red days. Over the course of the next 3 to 4 cycles, Lady-Comp learns your unique menstrual cycle rhythm.

Cycle calendar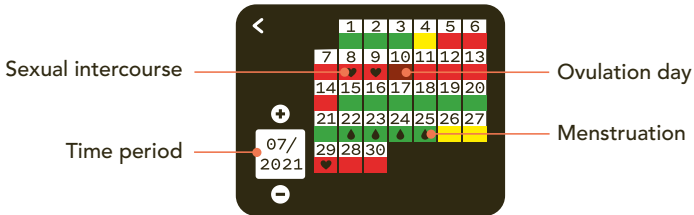

The cycle calendar shows you the fertility statuses of your cycles as measured for the last 720 days. You may also call up a menstruation prognosis for the coming 6 months.

Cycle curve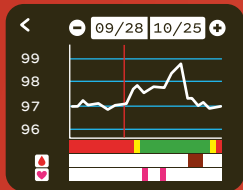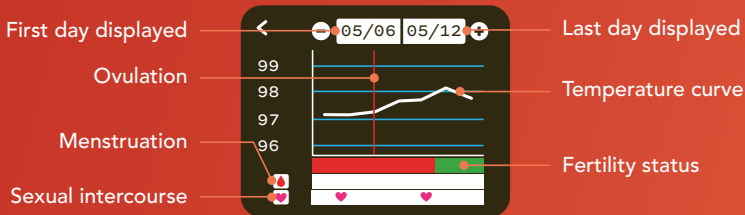


You can display a larger time period by pressing the circle button.


Press  Button: 1x 2x 3x

Display in days: 14 25 50


Cycle and planning statistics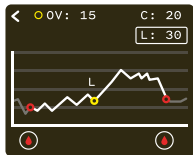


Average cycle length

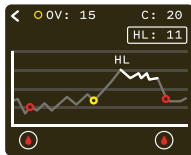Average length of high-temperature phase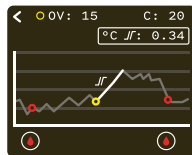

Average temperature increase after ovulation

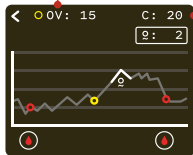Fluctuation range of ovulation in days

OV = Average cycle day of ovulation

C = Number of stored cycles


Date of conception

Estimated due date

Week of pregnancy


Planning Data

i Forecasts and statistics are only available from the beginning of the third cycle.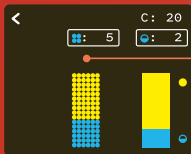


Number of cycles with CLI (corpus luteum insufficiency)

Corpus luteum insufficiency is characterized by a shorter high-temperature phase and/or a lower temperature increase. This indicator appears only after 3 cycles with a short high-temperature phase.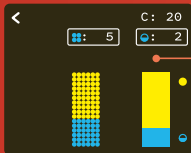

Number of monophasic cycles

Displays monophasic cycles. If no ovulation takes place in a particular cycle, the indicator will remain on red in this case.

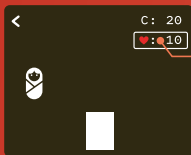Planning hits

The amount of sexual intercourse inputs that were consistent with the predicted ovulation and highly fertile days. This reflects the number of hits in the last 20 cycles and is only displayed if you have chosen to enter sexual intercourse information.

Create your fertility report

Turn the device counterclockwise and connect Lady-Comp with your computer via the USB cable. Copy the file *.txt into a local folder and load the file into the form displayed on our website lady-comp.com/analyser

To close the device, place the notches of the 2 parts of the device slightly offset on each other and turn the device clockwise.

Delete data

To delete all entered data, you have to press the minus button in the menu "Delete data" 10 times in a row. This will fill the circles of the display step by step. If the deleting is successful, you will hear a signal tone.

Cleaning

- Clean the sensor of Lady-Comp carefully with a damp cloth. You can also use a small amount of alcohol or soap.
- Make sure that only the mouthpiece comes into contact with liquid, as Lady-Comp is not waterproof.

Batteries

- 2 pieces, Micro AAA, 1.5 Volt or equivalent AAA batteries.
The batteries must be changed while Lady-Comp is turned off.
- Follow your local battery disposal instructions.

Maintenance and care

- Lady-Comp has a life span of 5+ years.
- Protect Lady-Comp from wetness and moisture to prevent damage.
Do not place Lady-Comp on / in a heat generator, such as a microwave oven, an oven or radiators.

Remedy and error messages


Check menstruation input. If Lady-Comp has not received a measurement or menstrual input for a longer period of time (> 4 days) and a new cycle start is expected during this period, this message reminds you to make up for any entries you may have forgotten.

i *If you had your menstruation during this period, add it before the next measurement. Enter menstruation as described on p. 16.*

Batteries are empty. The batteries are completely discharged. No further operation is possible with this display.

i *Replace the batteries. Unscrew the device in counter-clockwise fashion. Insert the batteries or rechargeable batteries and place the notches of the 2 parts of the device slightly offset on each other and turn the device clockwise to close it again.*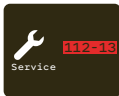

Error (Service). An error was detected.

i *Please contact customer service.*

Technical data

Name and model: **Lady-Comp**

Power supply: **Internal power source**

USB connection: **USB Mini**

Dimensions: **(H/W/D) 9/9/2.7 cm**

Weight: **130 g (inclusive batteries)**

Measurement accuracy

– **Device: 0.06 °C / 32.108 °F**

– **Sensor: 0.08 °C / 32.144 °F**

EAN Code: **7629999894334**

Contraindications

In the following cases, you should **not** use Lady-Comp:

- You suffer from a condition in which pregnancy could be dangerous for you, e.g. your blood pressure is too high or too low, you are suffering from heart disease, etc.
- Health risks exist during pregnancy for your unborn child, e.g. if you are suffering from drug or alcohol addiction, or if you are taking medication that may cause birth defects.
- If you are undergoing hormonal treatment, or if you are using hormonal methods of birth control.
- In case of very irregular bleeding. This can interfere with the prediction of your fertility or render it unusable.
- Lady-Comp is not a medical thermometer and should not be used to measure body temperature as an indication of fever or illness.

Note on electromagnetic compatibility


Electrical medical appliances are subject to special precautions regarding electromagnetic compatibility (EMC). Lady-Comp is intended for non-commercial home use only.

Portable and mobile RF communication appliances may affect medical electrical equipment. Lady-Comp meets the requirements of EN 60601-1-2 for electromagnetic compatibility. Details of this measurement data can be obtained from our Customer Support or from lady-comp.com

Target group profile

Women who are 18 years or older and can read and write.

Device label

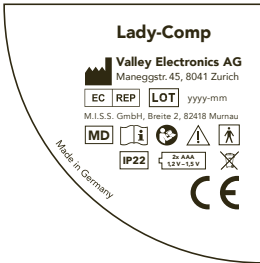The following figure shows the device label that is attached to the back of your Lady-Comp.

Service and help

In case of unforeseen events and problems during use, the manufacturer can be contacted at any time. A defective sensor can be re-ordered through the distributor in your country. If you have any questions about your cycle, please contact the distributor in your country. You can find all distribution partners on lady-comp.com

General information

Manufacturer

**Consult instructions
for use**

**Read these
instructions carefully**

Attention

CE label

Medical Device

**LOT number /
manufacturing date**

Power Supply
1.2 – 1.5 V (AAA)

WEEE: Please dispose of this appliance at public recycling depots for electrical appliances. Local laws/regulations prohibit disposing of this appliance and/or its battery with your general domestic garbage.

Type-BF application parts

IP code: Protected against solid foreign bodies with a diameter of 12.5 mm and splash-proof when the housing is tilted at 15°.

**EU Authorised
Representative**

Warranty information

Lady-Comp has been manufactured with the utmost care using premium-quality, state-of-the-art materials. Each Lady-Comp is subjected to numerous controls and inspections prior to delivery.

1. We guarantee the original purchaser that Lady-Comp is free of defects in materials and workmanship during the warranty period.
2. The warranty period begins on the day the new Lady-Comp is sold to the end user by an authorized dealer.* The warranty is valid for 2 years.
3. We undertake to either repair a defective device or replace it with a new one. Further claims, and, in particular, claims for losses and damages, including consequential damages, are excluded. The warranty does not cover indirect damages.

** Please note that our limited warranty applies only if you purchased your Lady-Comp from an authorized Valley Electronics dealer.*

4. The warranty does not cover any damage caused by improper handling, especially by forcible opening the device, or by other external influences.
5. Warranty claims are only valid if the user indicates the defect immediately in writing within 2 weeks after such a defect is discovered. Once the defect is discovered, the user should no longer use the Lady-Comp, but return it to the authorized distributor with a copy of the original purchase invoice via insured delivery.

If you have any questions or require further information, please do not hesitate to contact your Lady-Comp dealer.

Storage and operating conditions

Temperature limit: This device can be stored at temperatures between $-10\text{ }^{\circ}\text{C}$ and $+60\text{ }^{\circ}\text{C}$ / $-4\text{ }^{\circ}\text{F}$ and $140\text{ }^{\circ}\text{F}$.

Temperature limit: This device can be used without restriction at temperatures between $5\text{ }^{\circ}\text{C}$ and $40\text{ }^{\circ}\text{C}$ / $41\text{ }^{\circ}\text{F}$ and $104\text{ }^{\circ}\text{F}$.

Humidity limit: This device can be stored without restriction at a relative humidity up to 93 %.

Humidity limit: This device can be used without restriction at a relative humidity of between 15 % and 93 %.

Pressure limit: This device can be stored at pressures of between 700 hPa and 1060 hPa.

Pressure limit: This device can be subjected to pressures of between 700 hPa and 1060 hPa.

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Valley Electronics AG
VE Valley Electronics GmbH
M.I.S.S. GmbH
Valley Electronics LLC